Face Shield EP03, Model B

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Please remove the protective membrane on both sides of the shield.

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Valmistaja

Guilin Refine Medical Instrument Co. Ltd.

No.8-3, Information Industrial Park, High-Tech Zone, Qixing District, Guilin, Guangxi, 541004, P.R. China

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Huippu Group Oy, www.hg-hygienic.fi, Puh. +358 50 441 4539 / +358 50 4904065

Sertifiointi

CE-approved. Tested according to EN 166:2001 The EU type control is carried out by UL International (Netherlands) B.V. Westervoortsedijk 60, 6827AT Arnhem, The Netherlands. Notified Body no. : 2821

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[EN] INSTRUCTIONS

Purpose and protection

The product is intended to protect against drip contamination during the Covid-19 Pandemics.

Storage and minimum durability

Store the product in its carton. Avoid exposing the product to direct sunlight during storage.

Disinfection and cleaning

The product is not sterile. Clean with:

- Water: Lukewarm water with the addition of a neutral detergent. Rinse afterwards with clean water and dry with a soft sponge, wool cloth or similar.
- Disinfection: Surface disinfection (Ethanol 50-80%)

Never use sharp objects to close or dirt.

Storage between uses

The product is sensitive to scratches after the protective film has been removed. If possible, avoid direct sunlight as UV light shortens the life of the product.

Damage, maintenance and service

Scratched or damaged product should be discarded. Otherwise, the product requires no maintenance.

Reuse

The product is approved to clean and reuse. Under the right conditions, the product's service life is several months, even with frequent use. Depending on handling, the service life can be both shorter and longer. When the product loses its transparency or feels unhygienic, it should always be discarded. It is always up to the user to decide when to dispose of it.

Recycling

Sorted as plastic packaging.

Manufacturer

Guilin Refine Medical Instrument Co., Ltd.

No.8-3, Information Industrial Park, High-Tech Zone, Qixing District, Guilin, Guangxi, 541004, P.R. China

Importer and marketing

Huippu Group Oy, www.hg-hygienic.fi, Puh. +358 50 441 4539 / +358 50 4904065

Certification

CE-approved. Tested according to EN 166:2001 The EU type control is carried out by UL International (Netherlands) B.V.

Westervoortsedijk 60, 6827AT Arnhem, The Netherlands

Notified Body no. : 2821

Allergy

The product may cause an allergic reaction in susceptible persons.

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